

DIRECT BILIRUBIN

Spectrophotometric
DIAZOTIZED SULFANILIC

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

Instrument: **AU 400 / 640**

Reagent preparation

Working Reagent: Transfer the contents of one Reagent B-D vial into a Reagent A-D bottle. Mix thoroughly.
Stable for 20 days at 2-8 °C.

Instrument settings

Specific Test Parameters					
Sample:	Volume	20 μ L	Dilution	0 μ L	
Reagents: R1	Volume	200 μ L	Dilution	0 μ L	Reagent OD limit:
Wavelength:	Pri.	540	Sec.	660	First L -0.1 First H 0.1
Method:	END				Last L -0.1 Last H 0.1
Reaction slope:	+				Dynamic Range:
Measuring Point 1:	First	0	Last	16	L 0 H 15
Measuring Point 2:	First	0	Last	0	Correlation Factor:
Linearity:	%				A 1 B 0
No Lag Time:					On-board stability period:
7. None selected			L	H	
8. Out of range			0.00	0.25	
Panic Value	L		H		Unit: mg/dL
Decimal places: 2					
Calibration Specific					
Calibration Type:	AB	Formula:	Y=AX+B	Counts:	2
					Process: CONC.
Point 1:	Cal. N ^o	OD	CONC	Factor/OD-L	Factor/OD-H
	#		*	-9999	9999
* Enter the assigned standard value					
# User defined					