

# ALBUMIN

Spectrophotometric  
BROMOCRESOL GREEN

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

## Instrument: **AIRONE 200 (=AUTOHUMALYZER 900 S)**

### Reagent preparation

Reagent is ready to use.

### Instrument settings

UNITS	g/L
DECIMALS	0
REACTION TYPE	End Point
VOLUME SERUM	4
PLASMA	4
URINE	
REAGENT 1	400
REAGENT 2	0
ABS RANGE MIN.	0.0
ABS RANGE MAX.	2.000
LINEARITY LIMIT	70
REAGENT BLANK	Yes
CONTAMINATING	No
DIFERENTIAL	No
FILTER 1	620
FILTER 2	None
TIMES MIX 1	0.2
INCUBATION 1	120
LAG PHASE	2
MIX 2	0.0
INCUBATION 2	0
MEASURE	1
MEASURE TYPE	Std
FACTOR	0.0
NORMAL RANGE	35 – 50

Vers. 0511