

ALKALINE PHOSPHATASE (ALP)

Continuous-spectrophotometric
AMP BUFFER (IFCC)

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

Instrument: **AIRONE 200 (=AUTOHUMALYZER 900 S)**

Reagent preparation

Working Reagent: Pour the contents of a Reagent B bottle into a Reagent A bottle. Mix gently.

Stable for 2 months at 2-8°C.

Instrument settings

UNITS	U/L
DECIMALS	0
REACTION TYPE	Kinetic
VOLUME SERUM	8
PLASMA	8
URINE	
REAGENT 1	400
REAGENT 2	0
ABS RANGE MIN.	0.0
ABS RANGE MAX.	900
LINEARITY LIMIT	1200
REAGENT BLANK	No
CONTAMINATING	No
DIFERENTIAL	No
FILTER 1	405
FILTER 2	None
TIMES MIX 1	0.0
INCUBATION 1	60
LAG PHASE	20
MIX 2	0.0
INCUBATION 2	0
MEASURE	30
MEASURE TYPE	Factor
FACTOR	2764
NORMAL RANGE	26 – 117

Vers. 0502