

CALCIUM

Spectrophotometric

ARSENAZO III

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

Instrument: **AIRONE 200 (=AUTOHUMALYZER 900 S)**

Reagent preparation

Reagent is ready to be used.

Instrument settings

UNITS	mg/dL
DECIMALS	1
REACTION TYPE	End Point
VOLUME SERUM	8
PLASMA	8
URINE	3
REAGENT 1	500
REAGENT 2	0
ABS RANGE MIN.	0.0
ABS RANGE MAX.	600
LINEARITY LIMIT	18
REAGENT BLANK	Yes
CONTAMINATING	Yes
DIFERENTIAL	No
FILTER 1	578
FILTER 2	None
TIMES MIX 1	0.0
INCUBATION 1	160
LAG PHASE	3
MIX 2	0.0
INCUBATION 2	0
MEASURE	1
MEASURE TYPE	Std
FACTOR	0.0
NORMAL RANGE	9.0 – 10.7

Vers. 0207