

URIC ACID

Enzymatic-spectrophotometric
URICASE/PEROXIDASE

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

Instrument: **ALCYON 300 (=FALCOR)**

Reagent preparation

Reagent is ready to be used.

Instrument settings

Type	End Pt.	Primary Wavelength	550
Unit	mg/dL	High Wavelength	---
Number of decimals	1	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (μl)	200	Antigen Excess test (%)	NO
Second reagent vol. (μl)	0		
Sample Volume (μl)	5.0		
Lower normal value	2.4	Slope	1.00
Higher normal value	7.0	Intercept	0.00
Systematic dilution	1	Reaction delay	05:00
Dilution For Urine	10	Incubation	00:00
		Total	05:00
Low Reagent Blank	0.0000		
High Reagent Blank	1.3000		
Maximum Deviation	0.0200		