

# IRON

Spectrophotometric  
CHROMAZUROL B

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

## Instrument: **BS-300**

### Reagent preparation

Reagent is ready to use.

### Instrument settings

PARAMETERS							
N°	...	Test	Iron C	Method	End-Point	Direction	Ascend
Unit	µg/L	Decimal	0	Prim.Wavelength	630	Sec. Wavelength	700
Sample Volume	15	R1 Volume	300	R2 Volume	0	Line Limit	N/A
Incubation	--	Check Antigen	No	Response		Sample Volume	...
Reaction	0 -- 50	Substrate	N/A	Lower	--	Dilution	...
				Upper			
R1 Blank		Mix R Blank		Linearity		Full Name	...
Lower	...	Lower	...	Lower	10	Print Number	...
Upper	...	Upper	...	Upper	500		
Calibration							
Method	Linear	Std. 0	0				
		Std. 1	(*)				
		Std. 2	--				
		Std. 3	--				
		Std. 4	--				
		Std. 5	--				

(\*) Enter the standard value  
... Value entered by the user