

ALBUMIN

Spectrophotometric
BROMOCRESOL GREEN

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

Instrument: HITACHI 705

Reagent preparation

Reagent is ready to use.

Instrument settings

TEST CODE	# (ALB)
ASSAY CODE	END POINT 1
SAMPLE VOLUME	5
R1 VOL. (µL)	500 - ...
R2 VOL. (µL)	1 - 1 - NO
WAVELENGTH 1	700
WAVELENGTH 2	600
RGT. BLK. ABS.	0
RGT. BLK. CONC.	0
STD. CONC.	*
FACTOR	...
STD. ABS. ALLOWANCE	10 %
NORMAL RANGE L	35 g/L
NORMAL RANGE H	50 g/L
ABS. LIMIT (RATE)	0
TEST CODE	#
	# Data entered by the operator
	* Assigned value

Vers. 0610