

PROTEIN

Spectrophotometric
BIURET

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

Instrument: HITACHI 705

Reagent preparation

Reagent is ready to be used.

Instrument settings

TEST CODE	# (PROT)
ASSAY CODE	END POINT 1
SAMPLE VOLUME	7
R1 VOL. (µL)	350 - ...
R2 VOL. (µL)	1 - 1 - NO
WAVELENGTH 1	700
WAVELENGTH 2	546
RGT. BLK. ABS.	0
RGT. BLK. CONC.	0
STD. CONC.	*
FACTOR	...
STD. ABS. ALLOWANCE	10 %
NORMAL RANGE L	65 g/L
NORMAL RANGE H	80 g/L
ABS. LIMIT (RATE)	0
TEST CODE	#
	# Data entered by the operator
	* Assigned value

Vers. 0309