

CREATINE KINASE-MB

ImmunoInhibition

The application parameters comprised here constitute a guide to facilitate the validation of our reagents by the instrument. It is advisable to validate the use when there is any change in software or reagent versions.

Instrument: **ILAB 900 (=SHIMADZU CL-7200/7300)**

Reagent preparation

Working Reagent: Pour the contents of the Reagent B into the Reagent A bottle. Mix gently.
Stable for 15 days at 2-8°C. Working Reagent must be protected from light.

Instrument settings

TEST N°:		CURVE N°:	
TEST NAME	CK-MB	NORMAL RANGE MALE L - H	0 - 25
BAR CODE		FEMALE L - H	0 - 125
MEASURING METHOD	Rate	VALID RANGE LL - HH	3.0 - 330
CALIB. METHOD	1 point	CORRECTION CONST. 1,2	1 - 0
UNIT	U/L	W1 - ABS INC / DEC, LIMIT	Inc - 2000
MEASURING POINT	30 - 43	HEMOLYSIS LIMIT	-
PHOTOMETRIC METHOD	Double	ICTERIC LIMIT	-
WAVELENGTHS 1, 2, 3	340 - 410	LIPEMIC LIMIT	-
SAMPLE VOL. N, D, I	12 - 6 - 30	CALIB. REPLICATES	5
R1 V. - STIR MODE (Nor)	300 - Yes - 0	REPLICATE, SENS. LIMITS	-
R2 V. - STIR MODE	-	SLOPE - OFFSET LIMITS	-
N°. DECIMAL POINTS	0	PARTNER TEST, CONST.	-
SAMPLE TYPE	Serum	PROZONE INC / DEC, LIMIT	-
... Data entered by the operator		* assigned value	