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		A25	A15
GENERAL	Test name	CHOLESTEROL	CHOLESTEROL
	Analysis mode	endpoint mon.	endpoint mon.
	Sample type	serum	serum
	Units	mg/dL	mg/dL
	Reaction type	increasing	increasing
	Decimals	0	0
PROCEDURE	Replicates	1	1
	Name of assoc. constituent	-	-
	Type of reading	bichrom.	bichrom.
Volumes	Sample	3	3
	Reagent 1	300	300
	Reagent 2	-	-
	Washing	1.2	1.2
	Predilution factor	-	-
	Filters	Main	505
Reference		670	670
Reading 1		300 s	312 s
Times	Reading 2	-	-
	Reagent 2	-	-
	Postdilution factor	2	2
CALIBRATION	Type of calibration	multiple	multiple
	Calibrator replicates	3	3
	Blank replicates	3	3
	Calibration curve	-	-
OPTIONS	Blank absorbance limit	0.200	0.200
	Kinetic blank limit	-	-
	Linearity limit	1000	1000

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- Allain CC, Poon LS, Chan CSG, Richmond W and Fu PC. Enzymatic determination of total serum cholesterol. *Clin Chem* 1974; 20: 470-475.
- Meiattini F, Prencipe L, Bardelli F, Giannini G and Tarli P. The 4-hydroxybenzoate/4-aminophenazone chromogenic system used in the enzymic determination of serum cholesterol. *Clin Chem* 1978; 24: 2161-2165.
- National Cholesterol Education Program Expert Panel. Third report of the National Cholesterol Education Program (NCEP) Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (ATP III). NIH Publication. Bethesda: National Heart, Lung, and Blood Institute; 2001.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACC Press, 1995.
- Tietz NW. Clinical guide to laboratory tests, 2nd ed. Saunders Co, 1991.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Press, 1997.